

BrailleNote APEX

Quick reference guide

Main Menu Structure

The KeySoft Menus

The entry point to KeySoft is the Main Menu. You can display each of these items in turn, and select one.

When you have done this, you are generally presented with another menu. The system may extend to 3 or 4 levels of sub-menu. This structure is an important aspect of KeySoft. Items are grouped together for ease of access, so a huge range of functions can be found from one starting point, the Main Menu.

Main Menu Structure

Word Processor

- Create a Document
- Open a Document
- Emboss a Document
- Print a Document
- Setup Options

Planner

- Open Planner
- Emboss Planner
- Print Planner
- Setup Options
- KeyPlan Maintenance Utility

Address List

- Add Address
- Lookup Address
- Copy Addresses
- Emboss Addresses
- Print Addresses
- Setup Options

Email

- Write an Email
- Read Email
- Send and Receive Email
- Setup Options

Internet

- Address? Press ENTER for (...)
- List of Favorites in My Favorites Folder

Chat

- Start new Conversation
- Review Conversation
- My Availability Options

Media Center

- Media Player
- Record a Memo
- Playback Memos
- FM Radio
- Setup Options

Book Reader

- Read Book in which folder? Press ENTER for (...)
- List of folders on Flash Disk

Scientific Calculator

- KeyPlus

Database Manager

- Database to use? Press ENTER for (...)
- List of KeyBase files in KeyBase folder

Games

- Play which game?
- List of Game files in My Games folder

File Manager

- Directory
- Copy File
- Erase File
- Rename File
- Move File
- Protection
- Zip file
- Unzip file
- Translate File
- Folder Manager

Utilities

- Connect to Active Sync Synchronization
- Backup or Restore Files
- Pronunciation Dictionary
- Date and Time Set
- Install Keysoft extension Application Program
- Miscellaneous Options
- Language options
- Security
- Key Management

Terminal for Screen Reader

- Braille Terminal port?
- Currently (...)

Keyboard Learn

- Keyboard Learn Mode

For Information press Space with I

- Version Details

For the Options Menu, Press Space with O, anywhere



Apex Models

BrailleNotes models are available with 32 or 18 cells

BrailleNote Apex BT 32



BrailleNote Apex QT 18



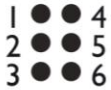
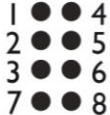
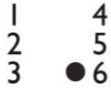

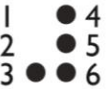
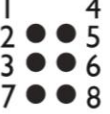
VoiceNote Apex QT























VoiceNote Apex BT



Braille Table

Braille grade 1 and 2	Computer Braille
 <p>The six dots of the Braille cell are arranged and numbered:</p>	 <p>In computer Braille, dots 7 and 8 are added.</p>
 <p>The capital sign, dot 6, placed before a letter, makes a capital letter.</p>	 <p>The capital sign, dot 7, placed before a letter, makes a capital letter</p>
 <p>The number sign, dots 3, 4, 5, 6, placed before the characters a through j, makes the numbers 1 through 0. For example: a preceded by the number sign is 1, b is 2, etc.</p>	 <p>In computer Braille, dots 7 and 8 are added. Dots 2,3,5,6,7,8 are used for numbers. Dot combinations for letters a to j, one row lower, make the numbers 1 through 0. For example, number 2 is obtained by pressing dot 2 and 3</p>

Braille Alphabet

a	b	c	d	e	f	g	h	i	j
									
k	l	m	n	o	p	q	r	s	t
									
u	v	w	x	y	z	capital sign	number sign	period	comma
